

**AVENT**  
Naturally


**AVENT**  
Naturally



**ISIS Breast Pump Nipple Protectors**

- |                              |                                     |
|------------------------------|-------------------------------------|
| • F: Tira-leit ISIS          | • Protège-mamelação Tendress        |
| • D: ISIS-Milchpumpe         | • Brusttrichter- Schmettbedingsform |
| • NL: ISIS Boortkolf         | • Topelboschecum                    |
| • E: Extractor de leche ISIS | • Protaga Pezones                   |
| • I: Tiralatte ISIS          | • Paraspinnoli in silicone          |
| • P: Boute Tira Leite ISIS   | • Protectors de mamilo              |
| • N: ISIS Brustpumpe         | • Brystsjold                        |
| • S: ISIS Bröstpump          | • Bröstvörtskydd                    |
| • PL: Laktator ISIS          | • Silikonowa osłonki piersi         |
| • RU: ISIS Молочкоотсос      | • Накладки на сосок                 |

AVENT®, Suffolk, CO10 7QS, England  
Tel. 44 1787 267 000 Fax. 44 1787 267 001  
[www.avent.com](http://www.avent.com)

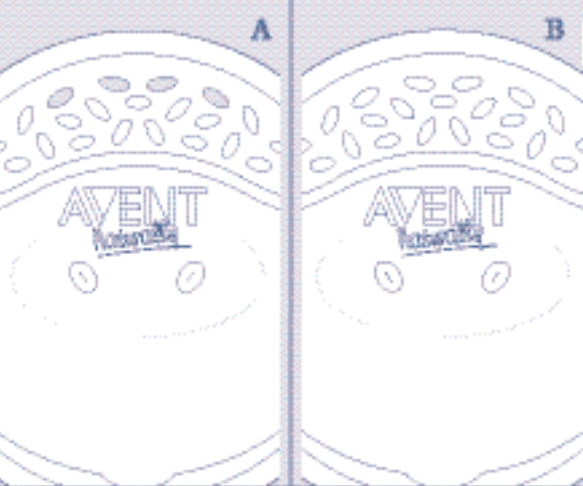

## AVENT ISIS Comfort Breast Shells with Ultrasoft Backing Cushions

Contents: 2 Ventilated Breast Shells, 2 Breast Milk Saver Shells (NO HOLES) and 2 Ultrasoft Backing Cushions

- A. Ventilated Shells** – to protect sore or cracked nipples
- B. Breast Milk Saver Shells** – to collect leaking breast milk
- C. Feet** – for stability
- D. Pouring spout** – for accurate pouring

The AVENT ISIS Comfort Breast Shells are worn inside the bra to help you overcome some common problems you may encounter during breast feeding.

### To protect sore or cracked nipples and help relieve engorgement


◆ Attach the Ultrasoft Backing Cushions to the Ventilated Shells and slip them inside your bra. The soft cushions will mould comfortably to the shape of your breasts and the domed shells protect your sore or cracked nipples from chafing. The ventilation holes allow the air to circulate.

◆ The Ventilated Shells can also be used to help relieve engorgement. They exert a gentle pressure around the areola and induce a slow leaking of your milk. DO NOT feed this milk to your baby, as milk collected over a long period of time may become contaminated.

### To collect breast milk for later use

◆ Sterilise the Breast Milk Saver Shells (no holes) and the silicone Backing Cushions in an AVENT Steam Steriliser (Electric or Microwave) or by boiling for at least 10 minutes.

◆ Attach the Ultrasoft Backing Cushions to the Breast Milk Saver Shells. While you are feeding or expressing from one side, slip an assembled Breast Milk Saver Shell inside your bra on the other side to collect any leaking milk. Only milk collected whilst feeding or expressing should be stored for later use.

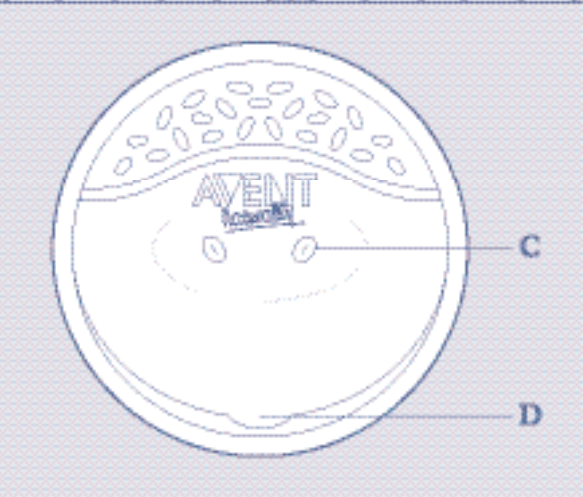

◆ ΕΟΡ&ΕΙΕΒΑ Ιη,ΙΕΑ ΟΒΕΙΕΙΟΙΟ,Α ,ΟΥ+,ΙΕ Ο Ι+ί+&ί+ΙΕ ΙΣΙΣ &ίη Ο·Ο+ ,Μ&ΙΟ,Ο ΙΟΙΟί+. ς+ ΟΟ &Αίη, ΟΟί+ ς' ΙΟ&ΙΕΒΑ &Α-ηί+ :Ο&ΙΟΕ ,Μ&... ,ΟΥ+,ΟΑ Ι+ί+&ί+Ο ΟΒΕΙΕΙΟΙΟ,Ο& ,ΟΥ+,ΙΟ& , ,·ΟΥ+,ί.ΟΑ& Ο &Μ,Ο& ΟΥ&Οί· ,·ΟΥ· Ο&·+;·Ο, Ο&Μ&Αί+ ,·ΑΑ ΙΟΙΟίΟ. Νήη Ο&Οί&Μ,·Α,Ο Ε&Ο&Ι,Α& ,+Ι&η ΙΟ&ΙΟ Ο&·+;Ιη. ΙΟΙΟίΟ, Ο&·+;ΙΙ&Α Ο&Ι,ΙΟ ,Ο, &Αίη ΙΟ&Ι&η Ε&η Ο·Α&Ε, +Ι&η.

◆ Ο&Ο· Ο&Ο&Ο&Ο-Ε& , Ι+ί+&ί+Ο, Ο&Ο&Ο&Ο&ΙΟ Ο&ΙΕΒΑ Ο&ΕΙΕΙΟΙΟ,Ο, ,ΟΥ+,ΙΟ, Ο&Α&Α&Β&Α Ο&·+;ΙΙ&Α ΙΟΙΟίΟ, Ο&Α&Ε, Ι·Ε Ο+Ι&Α Ε&η Ο&Ο&Ο&Α&Ε&Ε&Α, +Ιη. ·ΟΥ·Ι&·Ι Ε Ι+ί Ι&ΕΙΟ ··Ο&Α&Α Ο&Ι&Α&Ο&Ε&Α , Ι&Ο&Ε&Ι. Ι&Ι. Ε&Ο&Ι&Α ΙΟΙΟίΟ Ι&Ε&Α Ι+;Ι&Ε Οη , Ι&Ο&Ε&Ι, Ι&Ι&Α &Ο 24 ~+Ο&, , Ι&Ο&Α&Ε, Ι&Ε Ι+Ι&Α - &Ο 3 Ι&Οη·Α,·.

◆ Ι&Ο&Ι&Α&ς&ε ε&ε&ε&αί&ο ς&ε ε&ε&α&Ν&α&Ο&ς&ε&ε&α ε&ε&α& ε&ε&ε&α&ε&α&ς&α. ς&ε&α&α&ς&α&ο! ς&Α &Αί&Ο&Ι&Α&ε&Ο&Α&Ο&η Ι&Ο&Ε& , Ι+ί+&ί+Ε &Οί,·Α 40 Ι&ΕΙ&Ο&·Α&Α Ο&Α&Α· ,+ , Ο&Ι Ι+ί Ο&Ε Ι&Ο,Ο& Ο&Ι&Α ,+Ο, Ε&Α&Ε Ι&Α& &+ ,Ι&Α&Ε Ι+ Ι&Ο&Ι·Ι·Α Ο&Ο&Ο&Ι&Ε.

Ν&Μ,·Ε&Α Ο&Ο, +&· Ε&Α +Ο&Ο&Ο&Ε&Ι&Α&Ι&Ο+  
Ο&Ε&Ι+&Ι&Α&Ε&Ι&Ο&Ο&Α&Ε &ίηη ,·Μ&Ι&Ο,·Ο ,·Οί+&Ο&Ι&Ε, +Ι&η ΑVENT

◆ Ο&Ι&Ε Ι&Α&Ο&Ο&Α&ε&Ο& ,Αί&Ι&Ο Ο&Α&Α&ε Ι&Ο&Ι&Α&Ε&ΑΙ ς'·Ο,Ο& ,Ο&Α& ,·Ο&··Μ&ε , Ο&Α&Α&Ο&Ι&Α&Ι+ , Ο·Α&Ε&Α Ι&Α-Ο&Ι· Ο&Α Ι&Ο&Ι&Ε·Α&Ο& , Ο Ι&ΟΙ&Ο+ Ο&Ο&Ι· . Ι&ΟΙ&Ο&Ο&Ο&Ο+ ΙΣΙΣ.Ε&Μ&ε, Ο&+;Ι&Α& Ιη,·Α, Ε &Α·ηί+Ο ·Ο&Α&Ο Ι&Α,·Α Α&Α ,Αη&Ο·.

◆ Ο&Ι&Ε Ο ς+Ο ,Ο&Ο+ηηί+·Α Ε&η Ο&Ο&Α&Οί+ ,·Ε&Α&η Ο&Ο&Ι&Ε, Ε ς' Ε&Ο&·Ο· ,+Α&Α·Ο&Ι&Α&Ι&·Α Ο·Ο·Α&η&η Ο&Ε Ι&Ο&Ι&Α&Ε&Ε, Ε&Ο&Ο&Ι Α&Ο&Ε&Α Α+·Ε&Ο&Ι·Α Ι+ί+&ί+Ε Ι+ Ο&Ο&Ο&Ι ΑVENT, Ε&Α,Ο&Ο, Ι&Α&Ι·Α Ε&Α Ο&Ο&Ι&Ο, Ο ηη,Ι&Ο,Ο Ο&Ε&Ι&Οί+. +Ι&Ε+Ι, Ιηη Ο&Ο&Ι+ Ι+ί+&ί+Ο&ί Ο·Α&Ο&Α·Ε, Α&Ο Ο&Α&Οί·Ε Ι&Οί+;Ι&Ο Ο &Α·ηί+Οί ,Ο, &Αίη Ι&Ο&Ι&η&η.

◆ Ο&Ι&Ε Ο ς+Ο Ο&Ι&Ο&Ι&Ε&Α Ε&η ,Οηη&Ο&·Α Ο&Ο&Ι&Ε, Ε ς' Ε&Ο&·Ο· ,+Α&Α Ο&Μ&ε&Ι&Ο&Ο&Ε Ο&Ε Ι&Ο&Ι&Α&Ε&Ε, Ε&Ο&Ο&Ι Α&Ο&Ε&Α α&Ο&Α&Ι&Ο&Α Ο&Ο&Α&Ι Ο&Ο&Οί+ ΑVENT , Ο&Α·Α&Ε&Α Ι&Α&Ο&Οί. Ι&Ε Ι&ΕΙ&Ο& Ι&Α&Ο&Ο&Α&ε&Ο& ,Αί&Ι&Ο Ο&Α&Α&ε Ι&Ο&Ι&Α&Ε&Α. Ε&Ο&Ο&Ι Ο&Ε&Ι&Α&Ο Ι&Α&Ο·Ι&Ο&Ε&Ι&Ο. Ο&Ο&Ι, Ε &Α·Α&Ι&Ο·Ο&ε&Α&Ο Ι&Α,·Α Α,·Ο Α+Ι, +Ο&Ε&Ο·.